



1

## The staff member will ask you:

- Have you had your hearing tested before?
- Do you have any concerns about your hearing?
- Has anyone suggested you get your hearing checked?
- Do you wear, or has it been recommended that you wear hearing aids?

The staff member will set up the test for you.



2

## Practice

There are two test modes to select from - multi circle or single circle. The single circle mode is suggested for those with poor memory and/or low vision.



- select Start Trial under each mode. Press the circle when you hear the beep (tone).
- tap ONCE each time a beep is heard (one beep consists of a series of short beeps).
- the beeps will get quieter and quieter until it is hard to hear, this is normal, and you should tap even if you only think you heard a beep.

3

## Test

Select preferred test mode and start testing.

Note: those supporting testing can track progress using the bar at the top of the screen.



4



## Questions

You will be asked some questions at the end of the test and the staff member/carer can assist you in entering your responses.

5

## Review results

Your results will be shown on the device. The staff member will discuss the results with you.

